

Instructions for use



Product Name

Sterile Transfusion Sets for Single Use

Description

The transfusion sets products include protective cap, piercing device, drip chamber, filter for blood and blood components, tubing, flow regulator, injection site, male conical fitting, hypodermic needle. All parts and materials used in this can meet medical requirement. Sterilized by EO, it characterized with sterile, pyrogen-free, easy to use and safe.

Model and Class

STS-ACNF-11, STS-ACNF-21, STS-AFNF-11, STS-AFNF-21, STS-AFNF-12

STS-ACNC-11, STS-ACNC-21, STS-AFNC-11, STS-AFNC-21, STS-AFNC-12

Main structure and properties

This product is composed of protective cap, closure-piercing device, tubing, drip chamber, filter for blood and blood components, flow regulator, injection site(if any), air inlet, male conical fitting, and scalp vein set or hypodermic needle (if applicable).

Indications

Please pay attention to following instructions when you use it as a disposable medical instrument:

1. Open the unit packaging of the product, remove the protective cap of closure-piercing device, and insert closure-piercing device into the medicine bottle or medicine bag.

2. Hang the medicine liquid bottle or bag upside down, loosen the flow regulator, squeeze the drip chamber, and control the liquid level in the drip chamber at about 2/3. After exhausting the air in the tube, fill the entire tube with medicine liquid, and close the flow regulator.

3. If the transfusion set is equipped with an hypodermic needle, pull out the protective sleeve of the hypodermic needle, insert the hypodermic needle into the human vein, open the flow regulator, and adjust the delivery speed of the drug solution until the desired speed is reached, then the infusion can be carried out.

4. If the transfusion set is not equipped with an hypodermic needle, open the unit package of the hypodermic needle, install the hypodermic needle on the male conical fitting at the end of the transfusion set, and follow the steps of 2 and 3.

Intended purpose

Transfusion Set is used to administer blood from a container to a patient's vascular system through a needle or catheter inserted into a vein.

Intended user: Trained medical institution nursing staff.

Intended patient population: Patients who need intravenous blood

Contraindications

1. Newborns, prepubertal men, pregnant and lactating women use DEHP-containing infusion sets with caution;

2. DEHP-containing infusion sets should not be used for infusion of fat-soluble liquids and drugs such as fat milk;

3.This product is forbidden to inject drugs that are incompatible with PVC.

Warnings

1. This product has been sterilized by ethylene oxide, sterile, pyrogen free, valid for 5 years.
2. This product is forbidden to be used in high pressure infusion system.
3. This product is only used by trained doctors or nursing staff.
4. Check before opening, packaging damage, protective cover off, there are foreign bodies prohibited use.
5. This product is only for one-time use, immediately after opening, destroy after use.
6. Stored in the relative humidity is not more than 80%, no corrosive gas, clean environment.

Symbol

	Manufacturer		CE mark and Identification number of Notified Body
	Authorised Representative in the European community		Unique Device Identification
	Importer		Batch code
	Medical Devices		Date of manufacture, made in China
	Caution		Use by
	Consult instructions for use		Keep dry
	Do not reuse		Keep away from sunlight
	Sterilized using EO, single sterile barrier system		Fragile, handle with care
	Do not re-sterilize		Keep upright during transport
	Do not use if package is damaged		Catalogue number
	contains latex		latex free
	contains phthalate		phthalate free

	Do not use if the packaging is damaged. Follow the instructions carefully.			Protect from sunlight
	Temperature limits			luftfeuchte begrenzung

Incident Reporting

The user and/or patient shall report any serious incident that has occurred in relation to the device to Anhui Easyway Medical Supplies Co., Ltd and the competent authority of the EU Member State in which the user and/or patient is established.

Environmental protection

After using this product, the disposal of medical device waste complies with relevant regulations and environmental protection requirements.

Storage method and service life

- 1.This product is valid for five years from the date of sterilization.
- 2.This product should be protected from direct sunlight, rain and snow. Product should be stored in a relatively humid less than 80%, non-corrosive gases, cool, dry, well ventilated, and clean environment.



Anhui Easyway Medical Supplies Co.,Ltd
No. 8, Tianfeng Road, Yongfeng Industrial Area, Tianchang City, 239300 Anhui, China
www.easywaycompany.com



Shanghai International Holding Corp. GmbH (Europe)
Eiffestrasse 80, 20537 Hamburg, Germany
Tel:+49-40-2513175

Rev.:2.0 effective date: 8.28. 2025