

Instructions for use



Product Name

Disposable Irrigating and Feeding Syringes

Description

Disposable Irrigating and Feeding Syringes are is mainly used for feeding of oral terminals and flushing of surgical wounds.

Model and Class

IFS-HH, IFS-PR, IFS-SS

Principle of operation for device

The Irrigating and Feeding Syringes use the “liquid pressure difference” to finish the feeding of oral terminals and flushing of surgical wounds. Before drawing drug into the barrel, the plunger stopper is pushed to barrel bottom to exhaust the air and the nozzle have inserted into the Liquid food or flushing drug container to form a negative pressure by pulling the plunger stopper, when the plunger stopper is gradually pulled up, the pressure at the front end of the plunger stopper is less than atmospheric pressure, and the pressure of the drug liquid level is equal to atmospheric pressure. Under the effect of pressure difference, the liquid medicine is inhaled into the syringe barrel. After drawing into the barrel, the nutrient solution or fluid food is injected into the human mouth by manual bolus, or the sterile rinsing liquid is injected into the human surgical wound by manual bolus injection.

Intended purpose

Disposable Irrigating and Feeding Syringes are is mainly used for feeding of oral terminals and flushing of surgical wounds.

1) Intended user: Professional medical staff, include physicians, nurse, therapist and other relevant staff with medical science.

2) Intended patient population

- Age group: non-limited, include newborn to geriatric;
- Weight range: all
- Health: Patients who need to feed through the oral cavity or clean the wound.

Indications

1. Open the unit packaging from the tear or unseal opening, and take out the product;
2. If the product is equipped with nozzle cap, remove the nozzle cap.
3. Pull the plunger to suck in the liquid food or flushing drug.
4. the nutrient solution or fluid food is injected into the human mouth by manual bolus according to the clinical injection method.
5. or the sterile rinsing liquid is injected into the human surgical wound by manual bolus injection according to the clinical injection method.

Contraindications and Warnings:

- 1.It should be sterile and free from pyrogen before package opened;

2.It has a valid period of five years, Do not use if it is beyond valid date;

3.Discard after single use.

4.Do not use if the package is opened or damaged, or protective cap is aparted or foreign matter is found;

5.It should be used by special trained person, and the skin should be correctly sterilized;

【Symbol】

	Manufacturer		CE mark and Identification number of Notified Body
	Authorised Representative in the European community		Unique Device Identification
	Importer		Batch code
	Medical Devices		Date of manufacture, made in China
	Caution		Use by
	Consult instructions for use		Keep dry
	Do not reuse		Keep away from sunlight
	Sterilized using EO, single sterile barrier system		Fragile, handle with care
	Do not re-sterilize		Keep upright during transport
	Do not use if package is damaged		Catalogue number
	contains latex		latex free
	contains phthalate		phthalate free
	Do not use if the packaging is damaged. Follow the instructions carefully.		Protect from sunlight
	Temperature limits		luftfeuchte begrenzung

Incident Reporting

The user and/or patient shall report any serious incident that has occurred in relation to the device to Anhui Easyway Medical Supplies Co., Ltd and the competent authority of the EU Member State in which the user and/or patient is established.

Environmental protection

After using this product, the disposal of medical device waste complies with relevant regulations and environmental protection requirements.

Storage method and service life

1.This product is valid for five years from the date of sterilization.

2.This product should be protected from direct sunlight,rain and snow.Product should be stored in a relatively humid less than 80%, non-corrosive gases, cool, dry, well ventilated, and clean environment.

Manufacturer

Anhui Easyway Medical Supplies Co.,Ltd

Manufacturing address

No. 8, Tianfeng Road, Yongfeng Industrial Area, Tianchang City, 239300 Anhui, China

www.easywaycompany.com

Authorized representative and address

Shanghai International Holding Corp. GmbH (Europe)

Eiffestrasse 80, 20537 Hamburg, Germany

Tel:+49-40-2513175

Rev.:2.0 effective date: 8.28. 2025